



Assembly Instructions




Main components

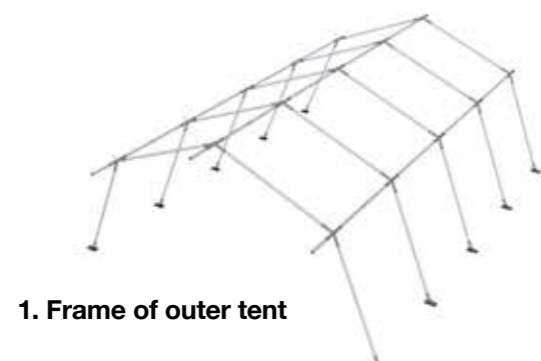


1. Frame of outer tent

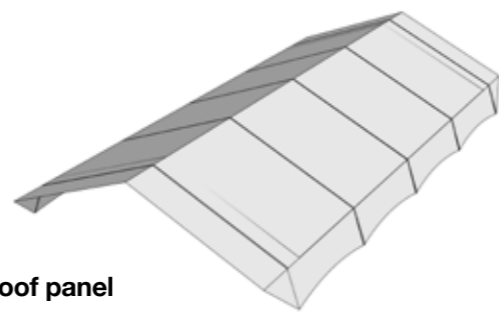

2. Roof panel

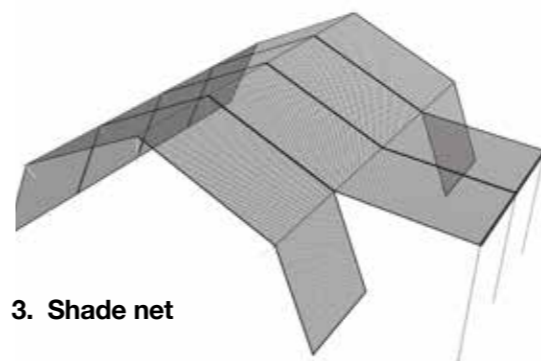


3. Shade net

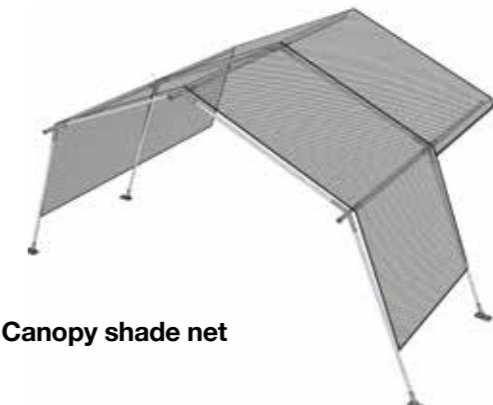

4. Canopy shade net



5. Large cabin (for confirmed patients)

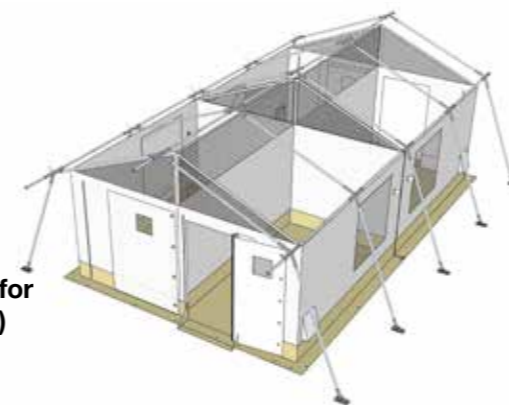




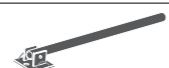





6. Individual cabins (for 4 suspected patients)

5 x i Ideal to assemble this legendmedi tent with single cabin or individual cabin in **45 minutes**


STAGE 1 Main frame roof panel and shade net


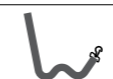
1. Frame of outer tent




A	22x	Roof poles	
B	6x	Extensions pole	
C	15x	Connectors	
D	10x	Standing poles with base plate	
E	4x set	Door sticks male and female	
	14x	TR pegs 35cm	
	4x	Rope 8mmx4m	
	1x	Hammer	
	1x	Assembly manual	



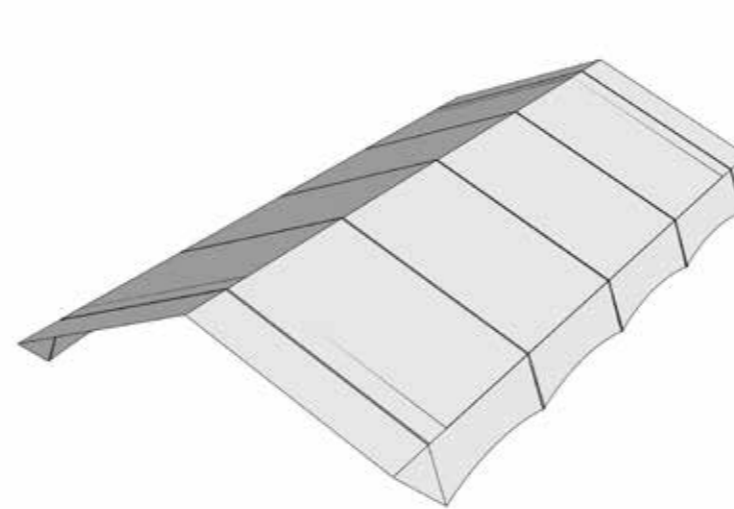

2. Roof panel

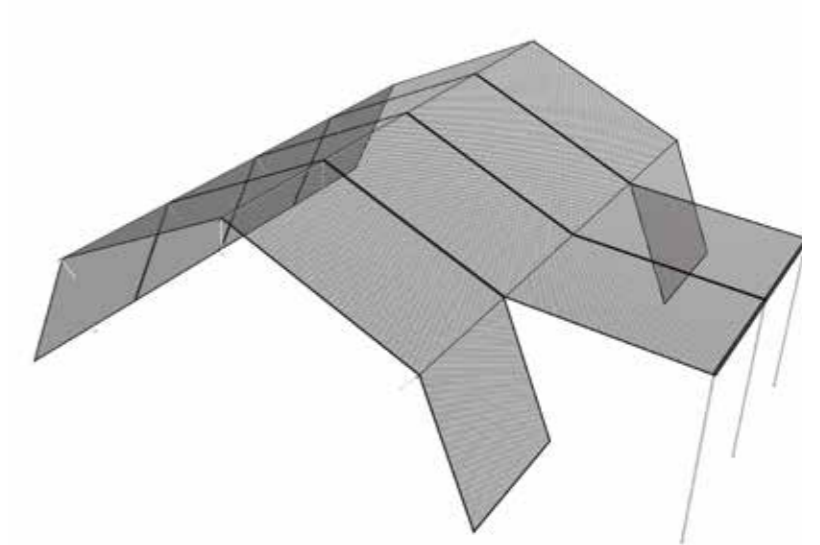


	1x	PE roof	
	10x	Webbing straps	

3. Shade net



F	1x	Shade net lateral including shade net frame	
	3x set	Sticks male and female	
	14x	TR Pegs 40 cm	
	1x	Hammer	

1

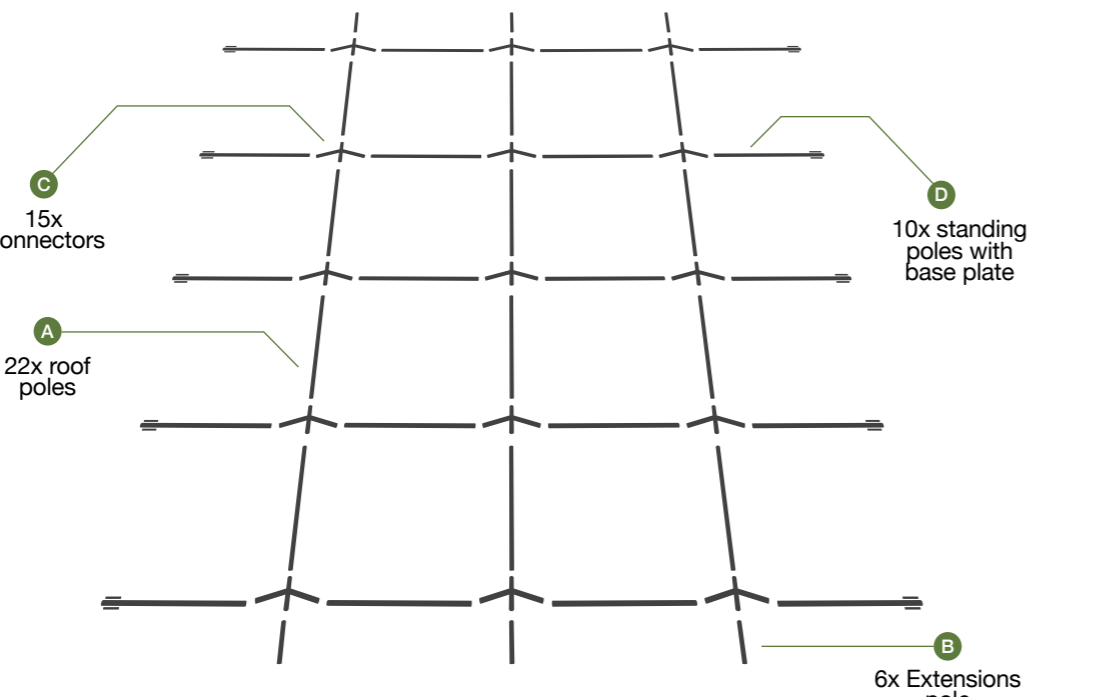

C 15x Connectors


A 22x roof poles

D 10x standing poles with base plate

B 6x Extensions pole

Layout all the poles and joints.

2

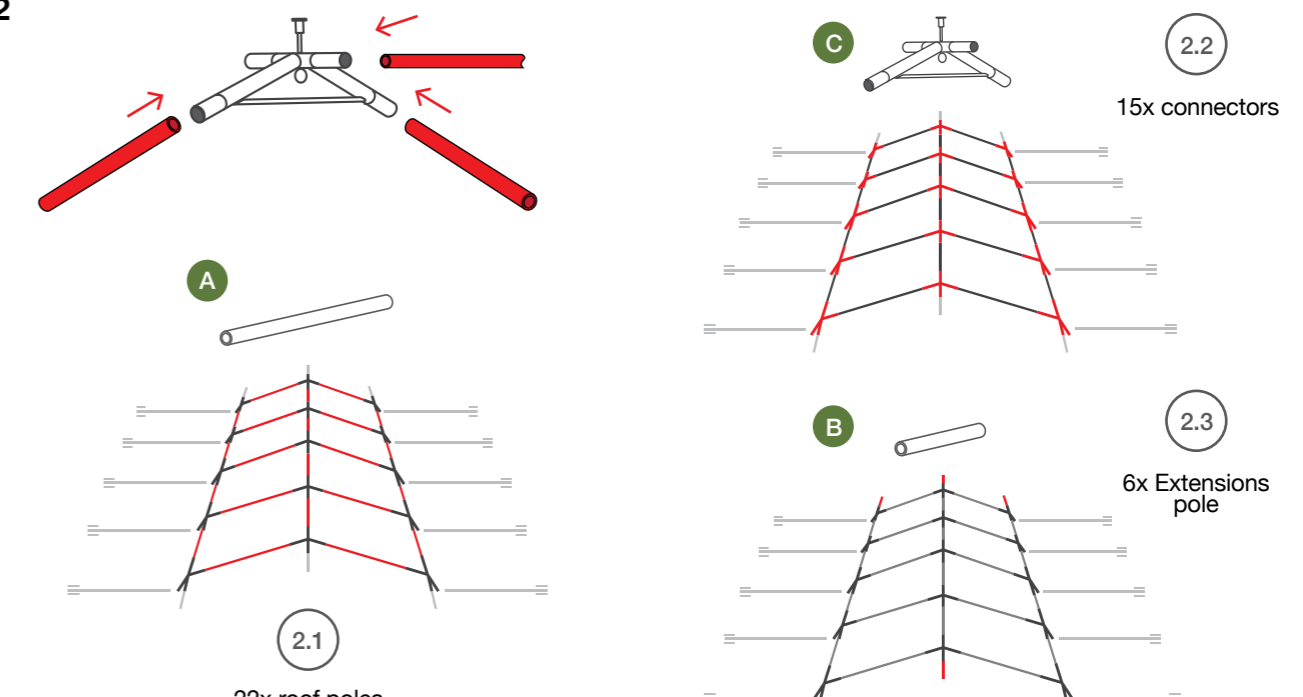


A 2.1 22x roof poles

C 2.2 15x connectors

B 2.3 6x Extensions pole


Slot all roof poles into the corner and ridge connectors. **Note:** the longer parts of the joint should face to the side.

3

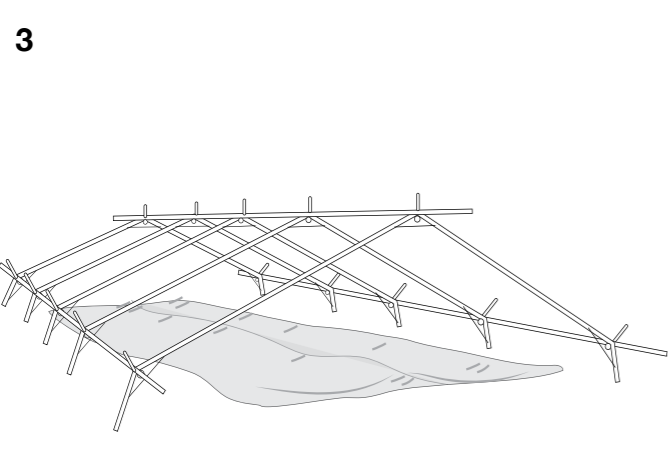


Unpack roof panel bag, position below the frame.

4

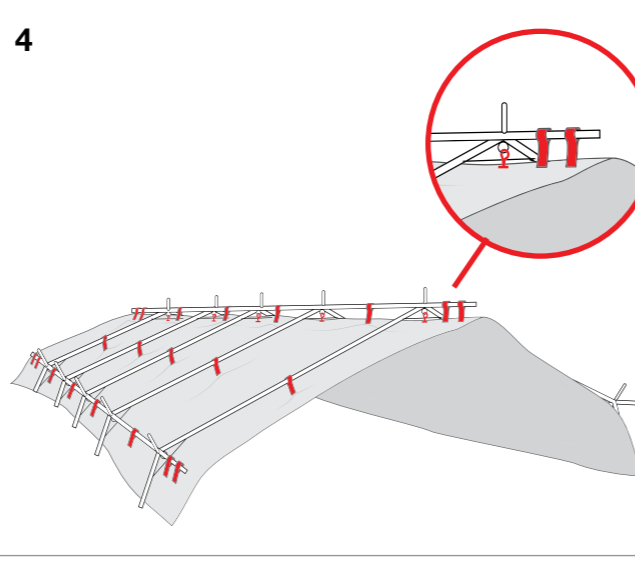

Attach roof panel to frame with Velcro straps and hooks.


5

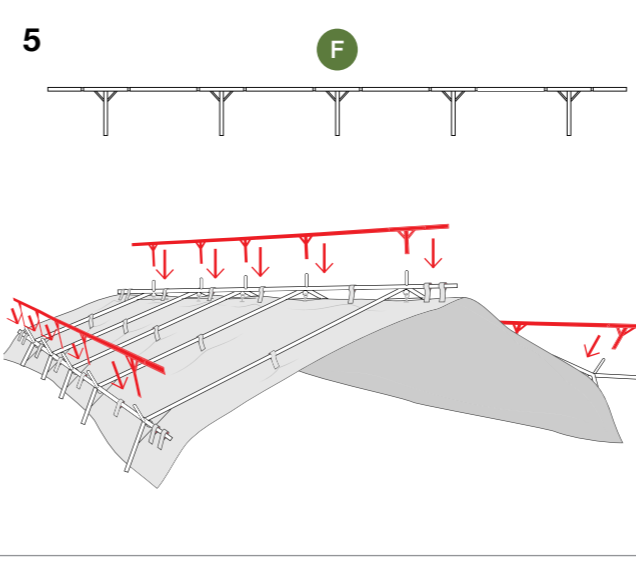

F


Unpack shade net bag and assemble identical shade net frames. Slot shade net frames onto ridge and sides of the frame.

6

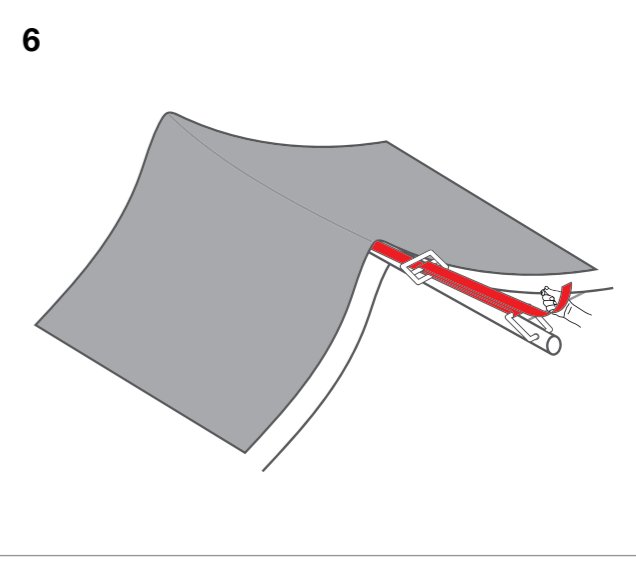


Spread the shade net over the top of the tent. Secure the shade net with the hook, tension using the buckle.

7

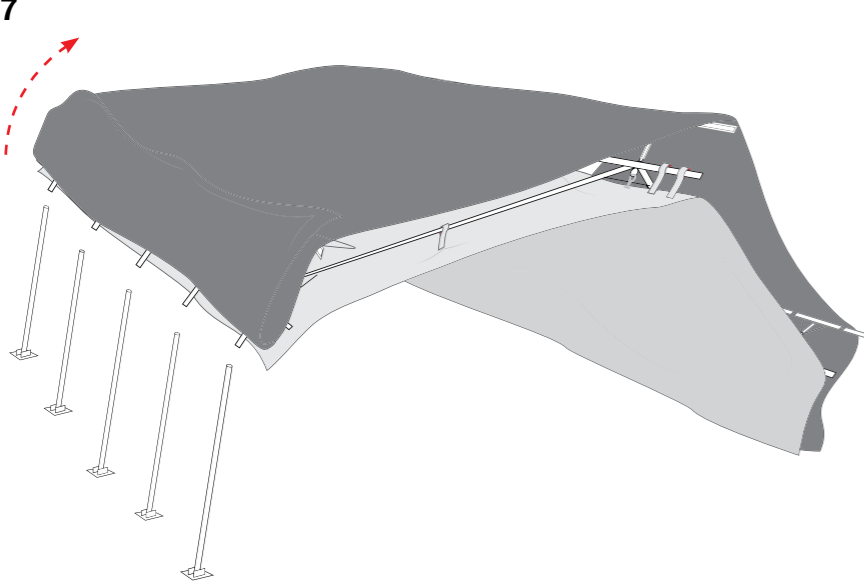


Lift frame and insert standing poles in connectors, start with one side only.

8

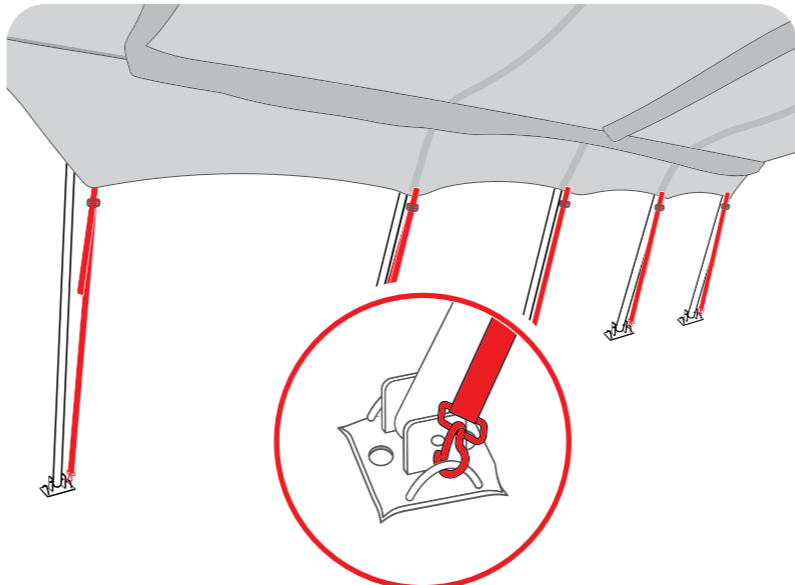


Hook black webbing straps into base plates.

9

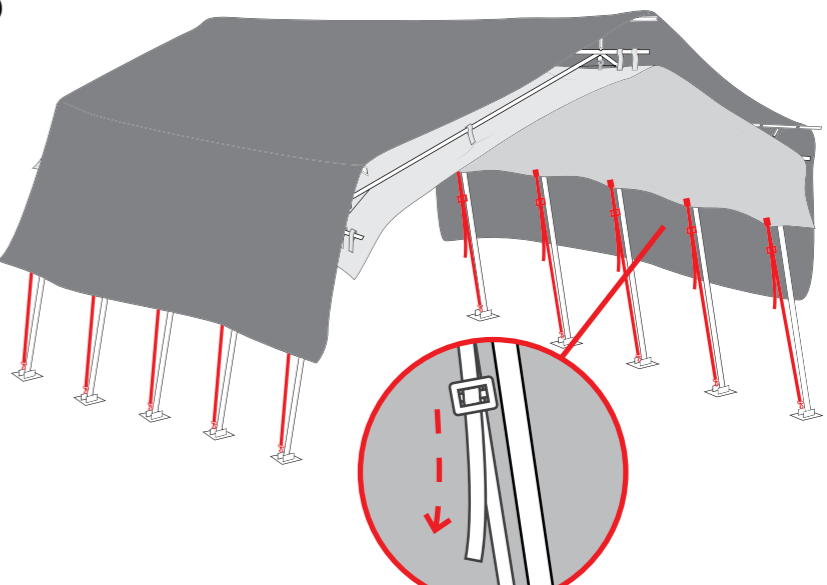


Lift one side after each other and tension the straps tightly.

10

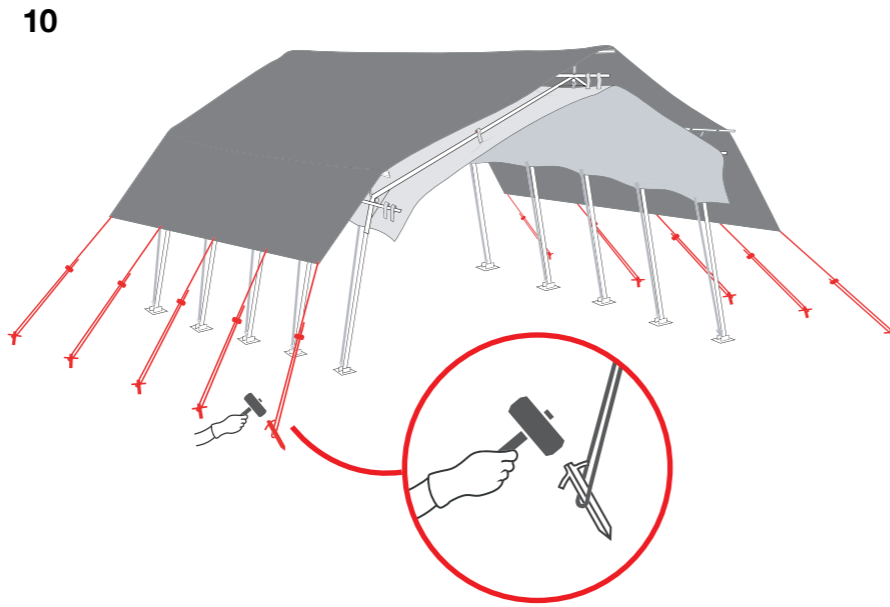

Pull out all guy ropes, secure with TR pegs and tighten with sliders.

11

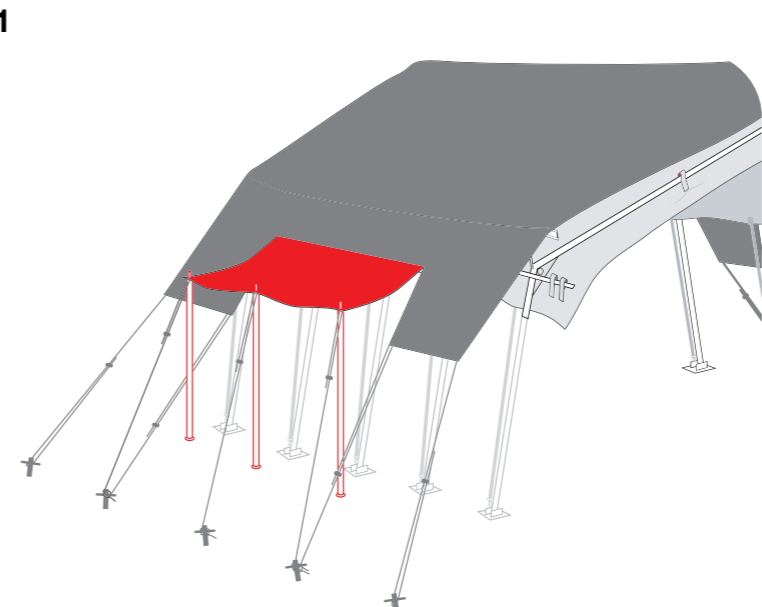


Lift middle section of shade net to create side canopy, if required. Use standing poles.

STAGE 2 Cabins for patients suspected with hemorrhagic fever

2 small isolation cabins, each with 2 treatment spaces





You can fix two individual small cabins to one LegendMEDI 45 tent. Each small cabin has the capacity to accommodate two suspected patients in separate treatment rooms.

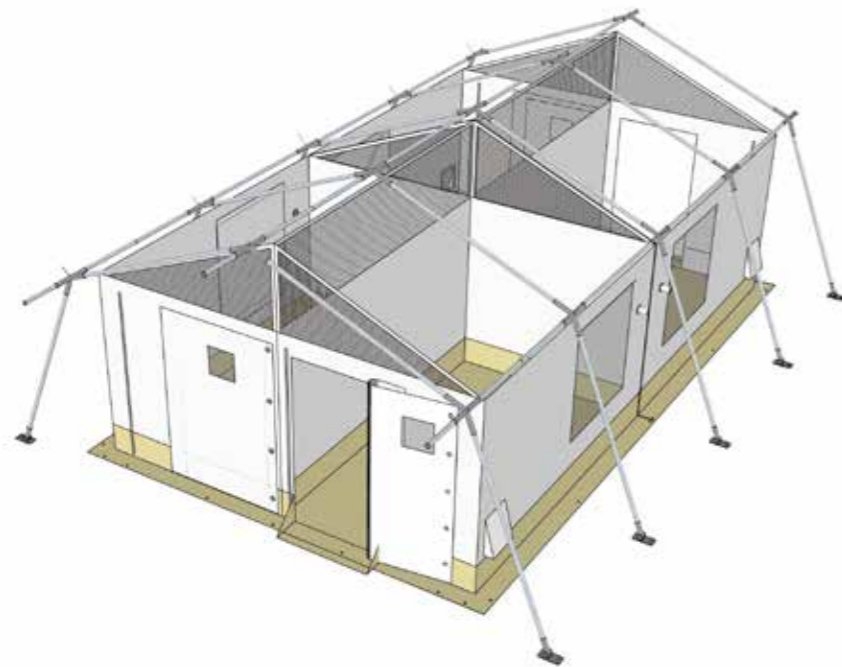

2x small cabins, each offering two treatment spaces




20x L-Shape pins 30 cm

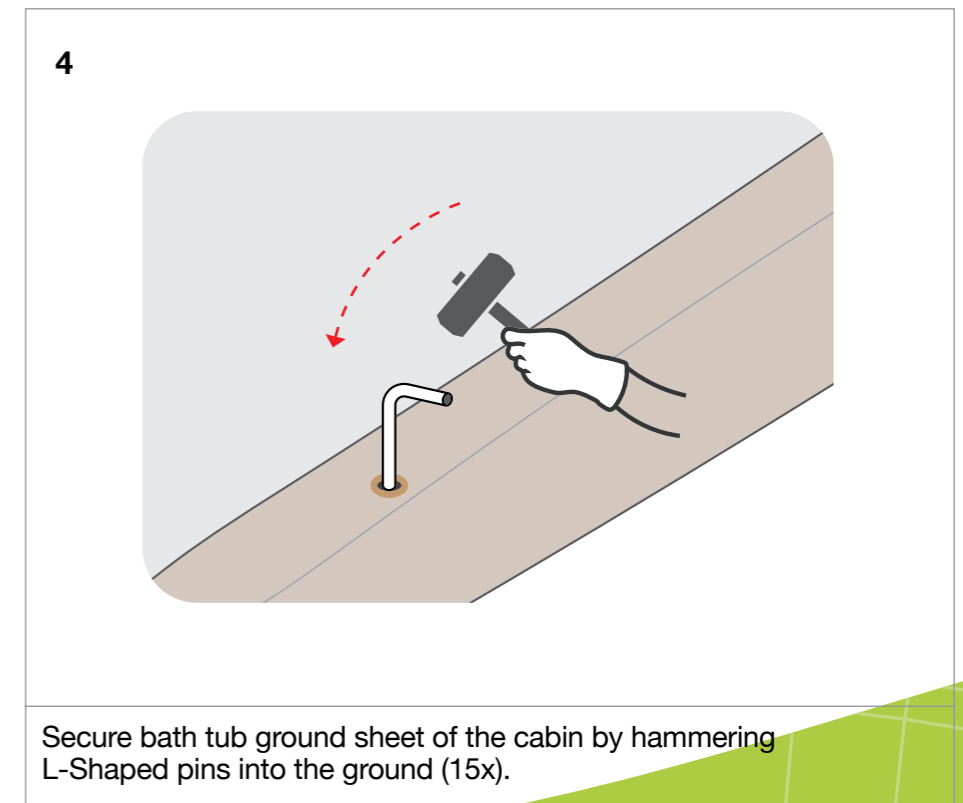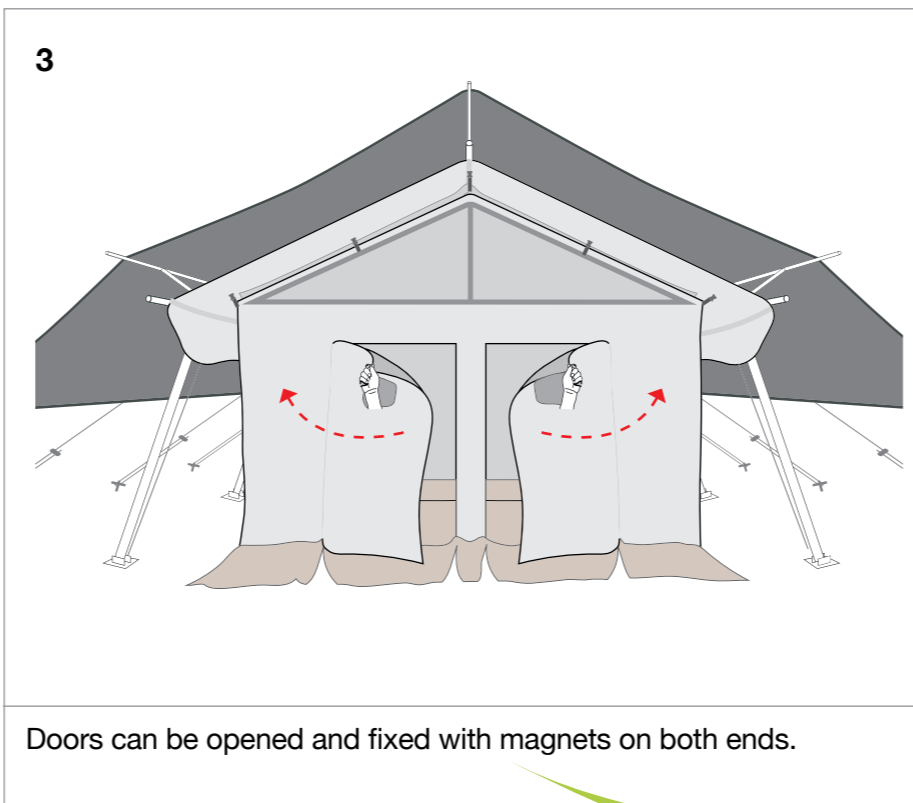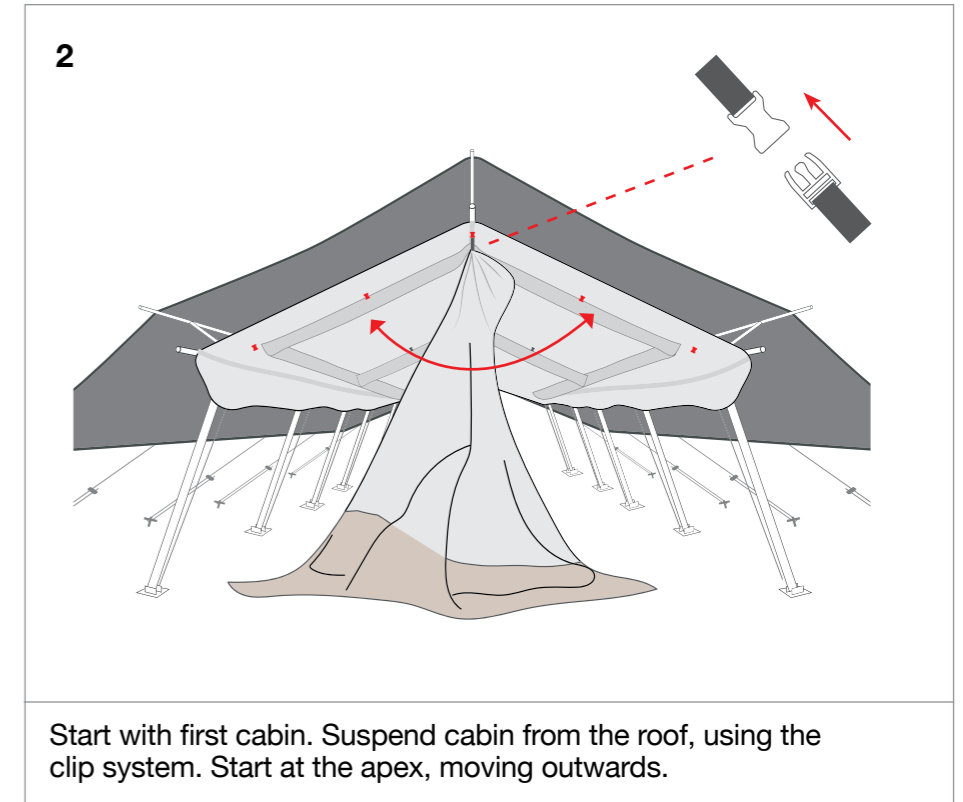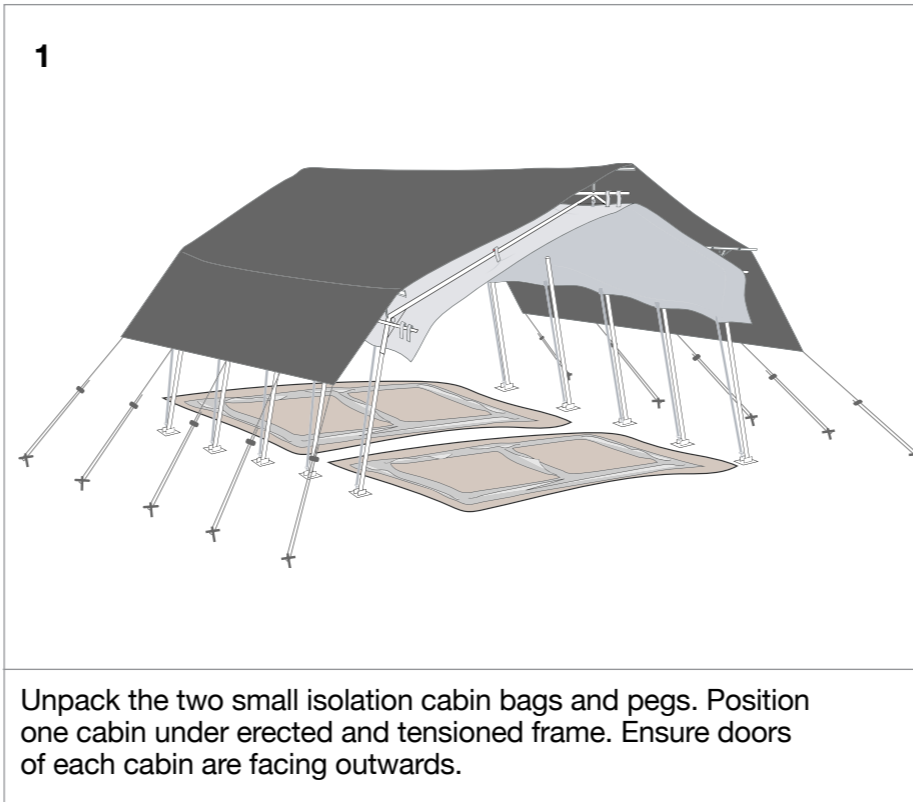

STAGE 3 Cabins for patients confirmed with hemorrhagic fever

Large isolation cabin, plus partition: for confirmed patients.

You can replace the two small cabins with one large cabin for confirmed hemorrhagic fever patients.

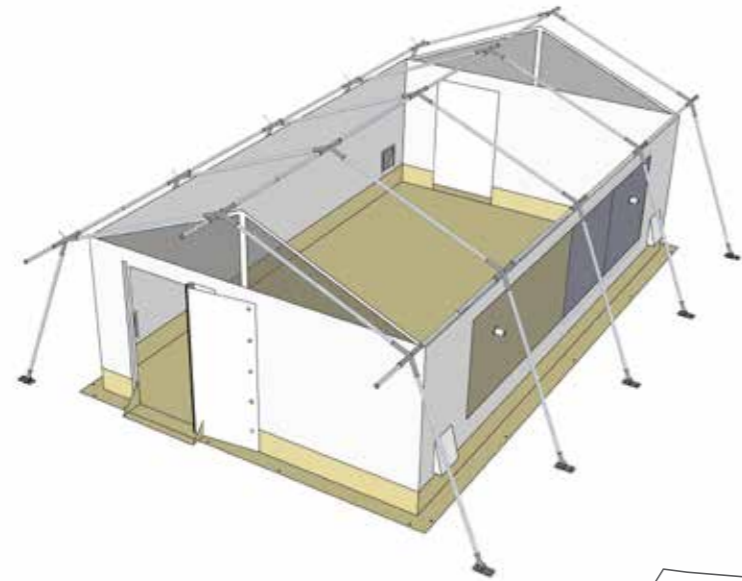

1x Single cabin




20x L-Shape pins 30 cm



1x Partition




1

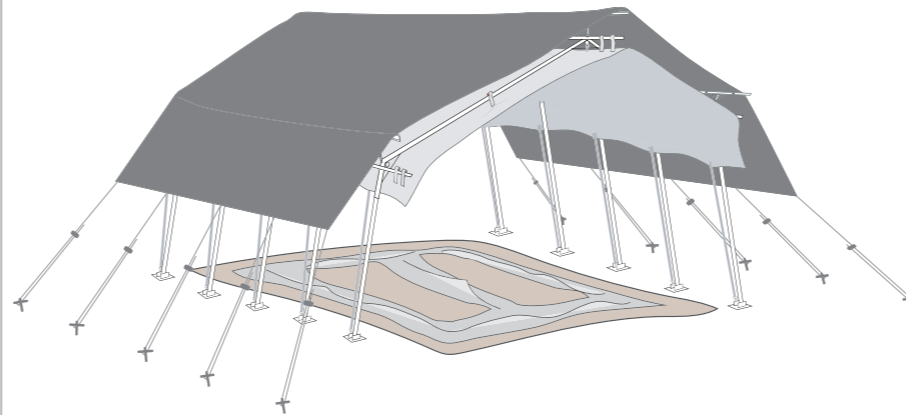


Unpack large cabin bag, position under erected and tensioned frame.

2

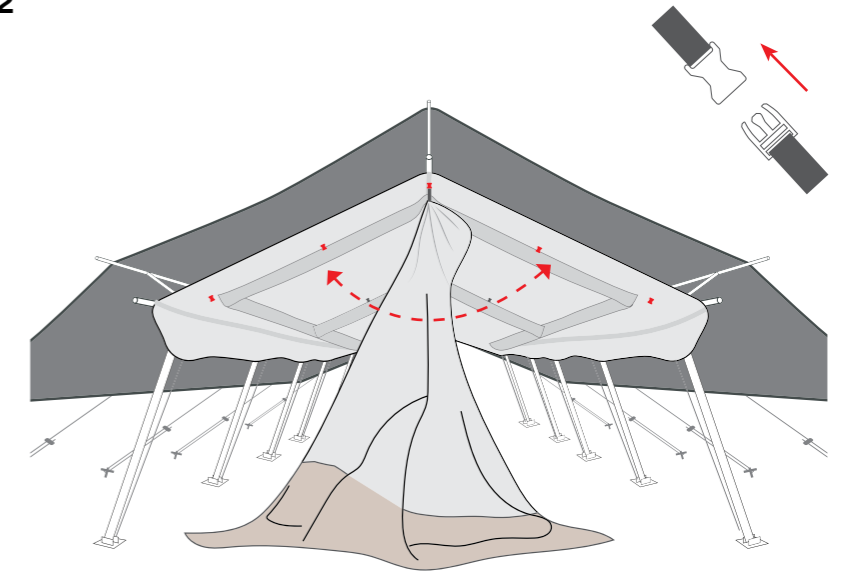


Suspend the cabin from the frame, using the designated clips. Start from the apex, in the middle.

3

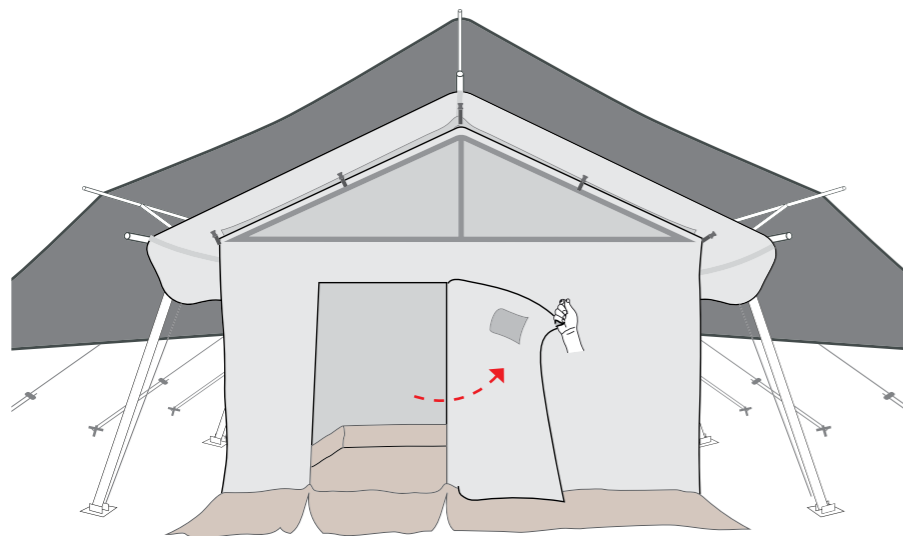


Open the door and fix with magnets on the right end.

4

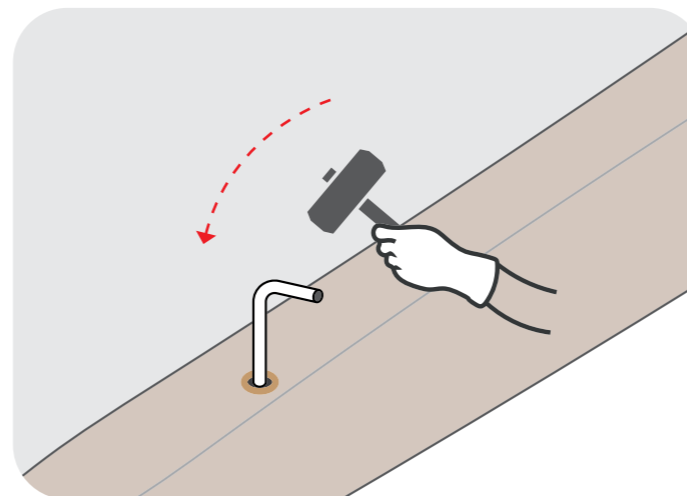

Secure cabin by hammering L-Shaped pins into the ground (30x).



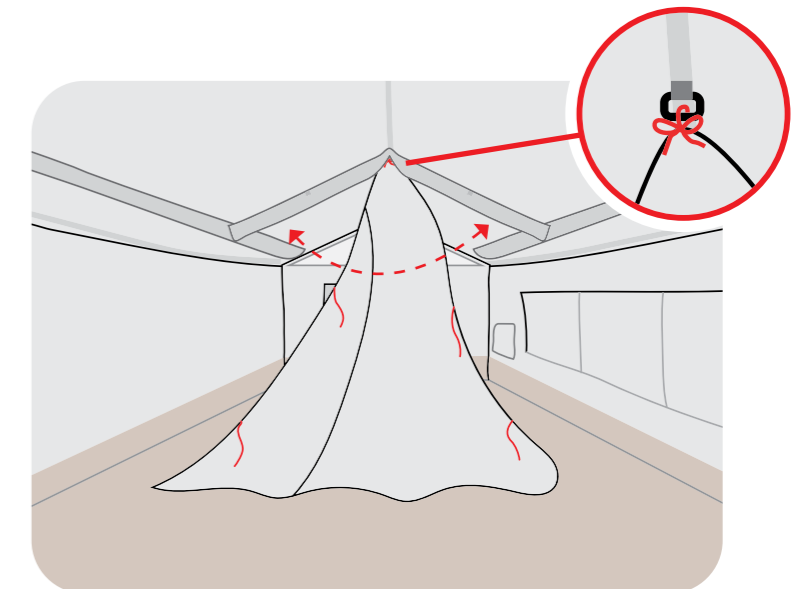



5




Unpack the partition bag and attach the partition to the roof by tying the strings.

STAGE 4 Optional Shade net canopy




1x	Shade net extensions including 3 arches of aluminum poles=1 set	
9x	Connectors	
6x	Standing poles with base plate	
12x	Roof poles	
6x	Pegs TR 40 cm	
1x	Hammer	

1

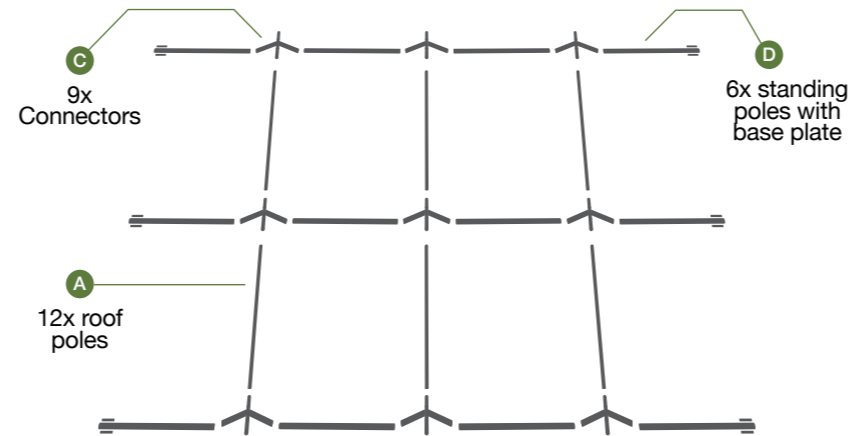


Layout all the poles and joints.

2

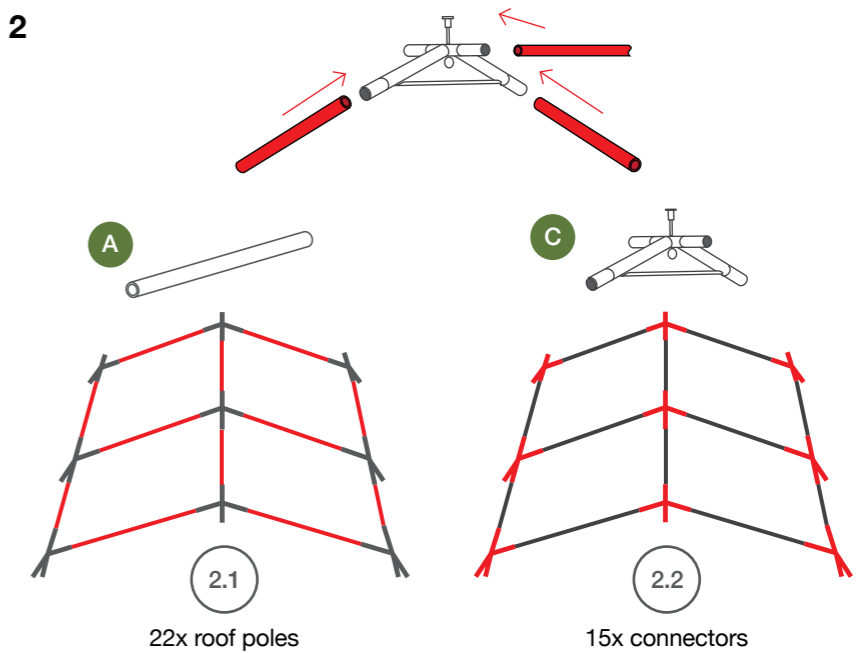


Slot all roof poles into the corner and ridge connectors.

3

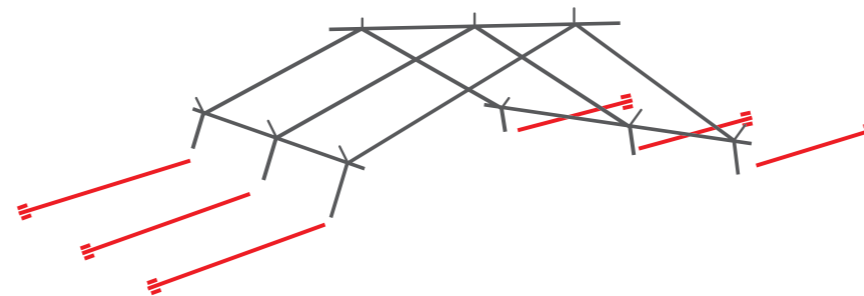


Arrange standing poles.

4

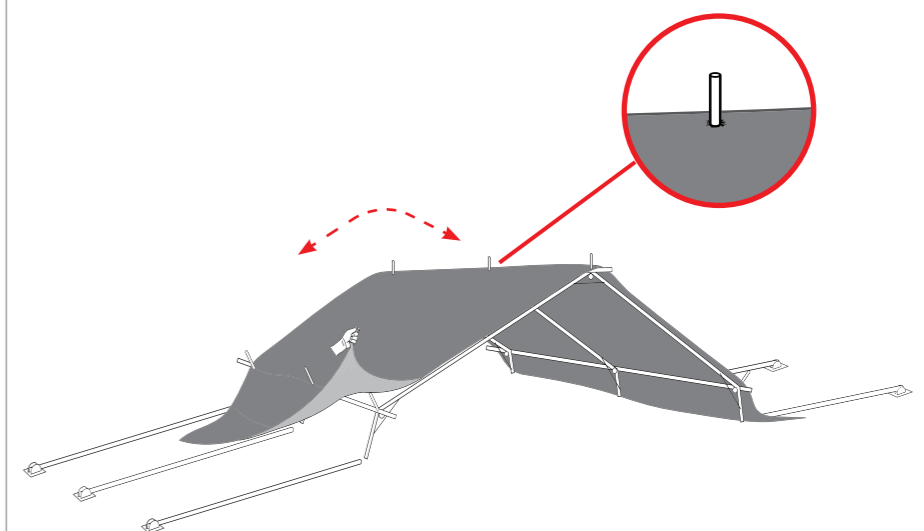


Spread the shade net over the top of the tent.

5

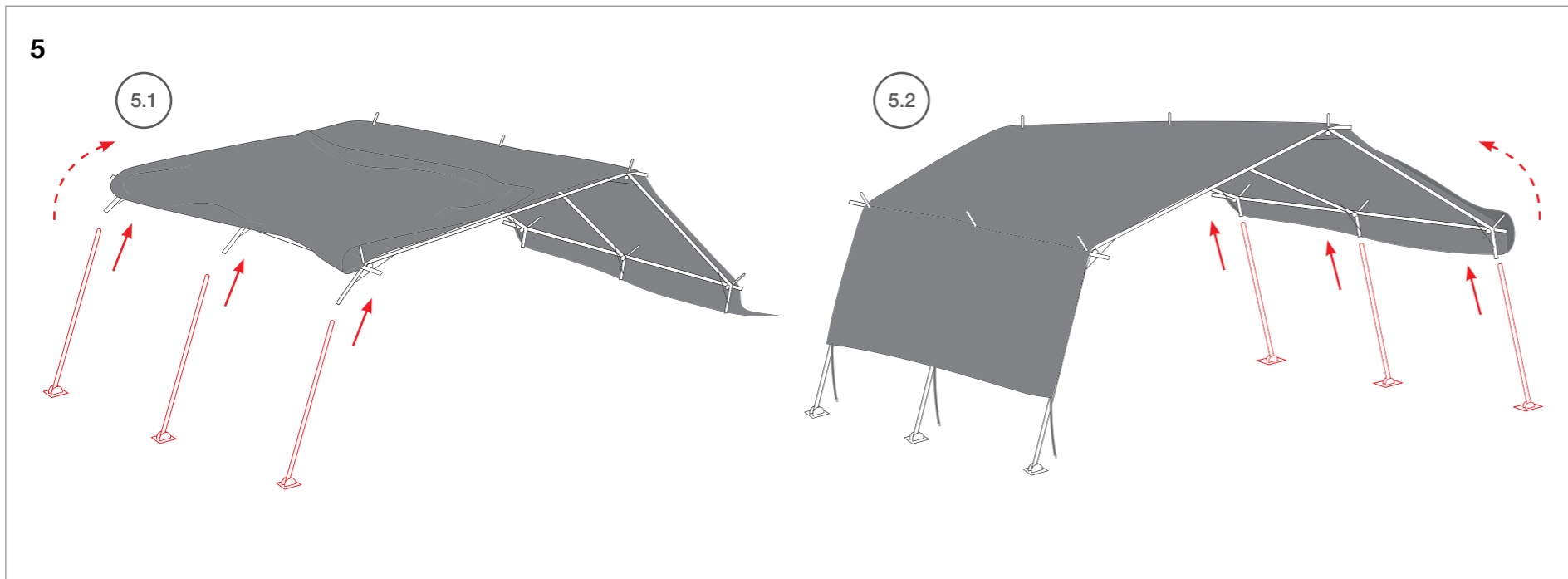


Lift frame and insert standing poles in connectors, start with one side and repeat to the other side.

6

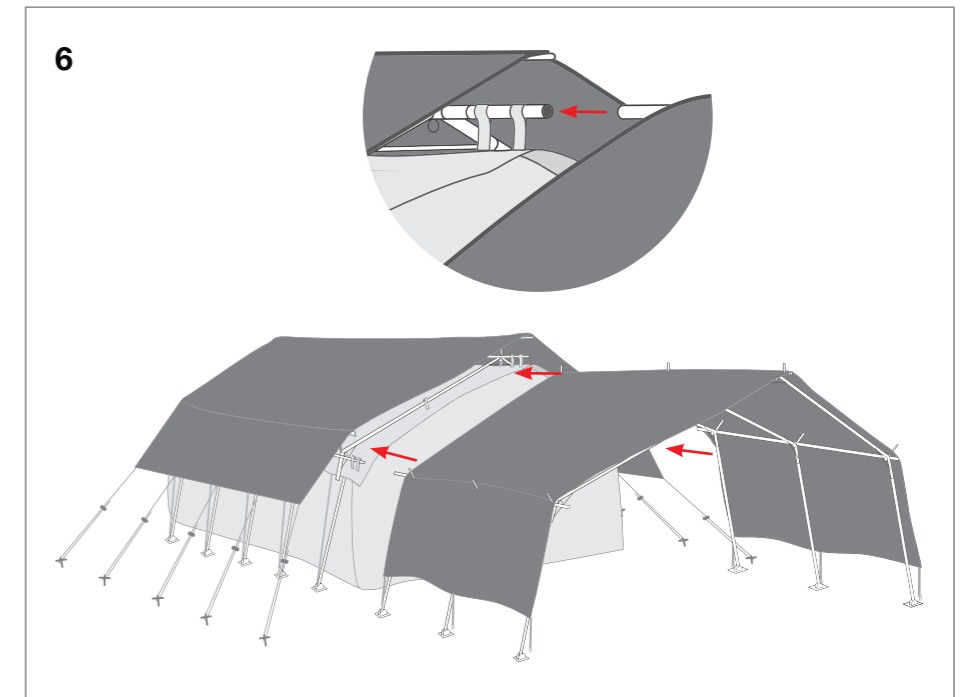

Join frame by connecting the systems together, slotting the pipes together.

7



Clip guy straps to the foot plates and tighten.